ARTWORK PANEL

3D Models

For more information, please call Matt Barrass on
+44 01482 710380 or e-mail 3D@sonoco-trident.com

Master Ref: SIP424722
Trident Reference No: **TR1893453**
SKU Description: Silhouette Soft Booklet
SMS22/23/25/26/100/102 COR BKT
Brand: Sinclair
Range: Silhouette
Product: Soft
Pack Type: Booklet
Pack Size: 5x2/3x2N/A
Action: **B**
Date: 27/05/20
Country: Corporate

CAD Ref: VP-BKT-88.9x254mm
Manufacturing Site: Vaupell
Printer: Aloha Graphics
Substrate: Art paper

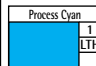
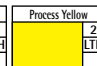
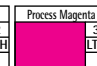
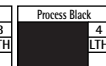
Barcode Type: N/A
Magnification: N/A
Pharmacode No/NE: N/A

Font Size (Main Body Copy): 6.00pt

Technical & Non Printing Items

 Cutter  Guides

Colours

Process Cyan	Process Yellow	Process Magenta	Process Black
 1 LTH	 2 LTH	 3 LTH	 4 LTH

TRIDENT

Connaught House, Connaught Road,
Kingswood Business Park, Hull, HU7 3AP, England.
T: +44 (0) 1482 828100

Please note that any low resolution paper Canon colour
copies associated with this job should be referred to for
content, layout and colour separation only.

UNDER NO CIRCUMSTANCES SHOULD THIS
ARTWORK BE ALTERED WITHOUT PRIOR
PERMISSION FROM TRIDENT.

88.9mm

254mm

SILHOUETTE SOFT®

Silhouette Soft® Products:

SMS22

SMS23

SMS25

SMS26

SMS100

SMS102

Instructions for Use

CE
2195

Sinclair Pharma US Inc.
18 Technology Drive Unit 134
Irvine, CA 92618, U.S.A.
Tel: +1 951 734 7701

EC REP

Sinclair France S.A.S
8ch Jubin, 69570 Dardilly
France
+33 4 72 52 07 60

ARTWORK PANEL

3D Models

For more information, please call Matt Barras on
+44 01482 710380 or e-mail 3D@sonoco-trident.com

Master Ref: SIP424722
 Trident Reference No: **TR1893453**
 SKU Description: Silhouette Soft Booklet
 SMS22/23/25/26/100/102 COR BKT
 Brand: Sinclair
 Range: Silhouette
 Product: Soft
 Pack Type: Booklet
 Pack Size: 5x2/3x2N/A
 Action: **B**
 Date: 27/05/20
 Country: Corporate

CAD Ref: VP-BKT-88.9x254mm
 Manufacturing Site: Vaupell
 Printer: Aloha Graphics
 Substrate: Art paper

Barcode Type: N/A
 Magnification: N/A
 Pharmacode No/NE: N/A

Font Size (Main Body Copy): 6.00pt

Technical & Non Printing Items

■ Cutter ■ Guides

Colours

Process Cyan	Process Yellow	Process Magenta	Process Black
1 LTH	2 LTH	3 LTH	4 LTH

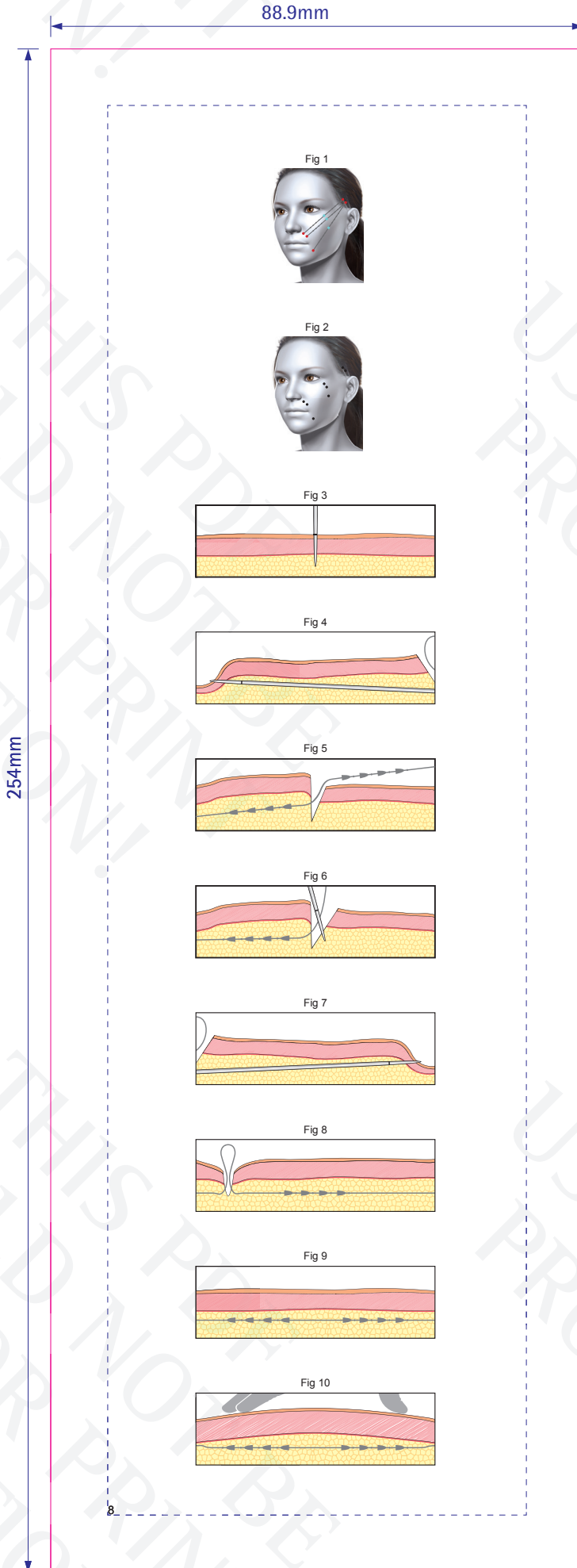
TRIDENT

Connaught House, Connaught Road,
 Kingswood Business Park, Hull, HU7 3AP, England.
 T: +44 (0) 1482 828100

Please note that any low resolution paper Canon colour
 copies associated with this job should be referred to for
 content, layout and colour separation only.

UNDER NO CIRCUMSTANCES SHOULD THIS
 ARTWORK BE ALTERED WITHOUT PRIOR
 PERMISSION FROM TRIDENT.

88.9mm

ARTWORK PANEL

3D Models
For more information, please call Matt Barrass on
+44 01482 710380 or e-mail 3D@sonoco-trident.com

Master Ref: SIP424722
Trident Reference No: TR1893453
SKU Description: Silhouette Soft Booklet
SMS22/23/25/26/100/102 COR BKT
Brand: Sinclair
Range: Silhouette
Product: Soft
Pack Type: Booklet
Pack Size: 5x2/3x2N/A
Action: B
Date: 27/05/20
Country: Corporate

CAD Ref: VP-BKT-88.9x254mm
Manufacturing Site: Vaupell
Printer: Aloha Graphics
Substrate: Art paper

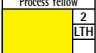
Barcode Type: N/A
Magnification: N/A
Pharmacode No/NE: N/A

Font Size (Main Body Copy): 6.00pt

Technical & Non Printing Items

 Cutter  Guides

Colours

Process Cyan	Process Yellow	Process Magenta	Process Black
 1 LTH	 2 LTH	 3 LTH	 4 LTH

TRIDENT

Connaught House, Connaught Road,
Kingswood Business Park, Hull, HU7 3AP, England.
T: +44 (0) 1482 828100

Please note that any low resolution paper Canon colour
copies associated with this job should be referred to for
content, layout and colour separation only.

UNDER NO CIRCUMSTANCES SHOULD THIS
ARTWORK BE ALTERED WITHOUT PRIOR
PERMISSION FROM TRIDENT.

EN

CONTENTS OF THE CARTON:

SMS22: 5 packs of SMS 22-PL-3.0.1-NA (2 units), 5 x patient chart labels, 1 x Instructions for Use,
SMS23: 5 packs of SMS 23-PL-3.0.1-NA (2 units), 5 x patient chart labels, 1 x Instructions for Use,
SMS25: 5 packs of SMS 25-PL-3.0.1-NA (2 units), 5 patient chart labels, 1 x Instructions for Use,
SMS26: 5 packs of SMS 26-PL-3.0.1-NA (2 units), 5 patient chart labels, 1 x Instructions for Use,
SMS100: 2 packs of SMS 22-PL-3.0.1-NA (2 units) plus 1 pack of SMS 23-PL-3.0.1-NA (2 units),
3 x patient chart labels, 1 x Instructions for Use,
SMS102: 3 packs of SMS 22-PL-3.0.1-NA (2 units), 3 x patient chart labels, 1 x Instructions for Use,

NOTE: All product variants are listed. Not all variants are marketed or approved in every territory.

INDICATIONS: United States: For export only. Not cleared for use within the USA.

International: Silhouette Soft is a resorbable sterile implantable single-use device intended to be used in adult patients for facial and neck reconstruction in the treatment of facial morphological asymmetry and ptosis, including but not limited to:
- the effects of facial ptosis or paralysis (e.g. Bell's palsy, benign or malignant tumours, iatrogenic injury, Varicella-zoster virus associated facial palsy, trauma, and congenital palsy),
- morphological asymmetry (e.g. congenital anomalies, trauma to the face or the temporomandibular joint).

DESCRIPTION:

Silhouette Soft is a resorbable implantable suspension suture for facial tissue lifting, consisting of a resorbable sterile monofilament USP 3-0/EP 2 size Poly L-Lactic Acid with molded cones made of a resorbable material, Poly (L-lactide-co-glycolide) 82:18, affixed to it. The 30cm device (SMS 22-PL-3.0.1-NA) has 8 cones, 27.5 cm device (SMS 23-PL-3.0.1-NA & SMS 26-PL-3.0.1-NA) has 12 cones and the 26.8 cm device (SMS 25-PL-3.0.1-NA) has 16 cones. All have two 12 cm, 23G straight needles attached to each end.

All products are supplied sterile (EO) for single use only.

ACTIONS:

Silhouette Soft elicits a minimal acute inflammatory reaction in tissue that is followed by gradual encapsulation. The device is for placement in subcutaneous or soft tissue.

PRECAUTIONS FOR USE:

The physician should be familiar with recommended techniques involving Silhouette Soft as well as proper patient selection and device placement. Ensure procedure is carried out in a sterile environment. Knowledge of anatomy of the site to be treated and specific precautions are essential in order to avoid damage to fragile structures such as nerves and blood vessels. Patients with extensive skin laxity, thin skin and/or severe malar fat sagging are not good candidates for this procedure. Take extra precautions for patients with neurological conditions. Take extra care with patients who have bleeding disorders and with patients who are taking medications that inhibit blood clotting (e.g. vitamin E, acetylsalicylic acid and salicylic acid derivatives and nonsteroidal anti-inflammatory drugs).

CONTRAINDICATIONS:

Patients with foreign body sensitivity or known or suspected allergies to implant or instrument materials in particular plastic/biomaterial should not undergo the procedure. Do NOT use in patients with active sepsis or infection, active (or history of) autoimmune disease, patients under 18 years of age, pregnant or breastfeeding women, or patients with limited ability of un-willingness to follow post-treatment recommendations.

INSTRUCTIONS FOR USE:

1. Ensure patients face and neck is make-up free, and clean the treatment site with antiseptic.
2. Mark the entry and exit locations where the Silhouette Soft suspension sutures are to be inserted (Fig 1 shows example placement for mid face treatment).
3. Remove sutures from protective pouch and place in sterile field using aseptic procedures.
Warning: Sutures are double pouched and only the internal surface of the inner pouch is sterile.
4. Administer appropriate local anesthetic on entry and exit points (Fig 2 example for mid face treatment).
5. Make a pre-incision hole with an 18G needle (not supplied) in the treatment point marked for insertion.
6. Insert one of the suture needles through the pre-incision hole to the required depth in the subcutaneous tissue as indicated by the black mark on the needle (Fig.3) and advance along the pre-marked location for each suture placement exiting the skin in the desired area (Fig 4). Run the suture through the skin by "pinch traction".
7. The needle exits through the pre-marked exit point and then the needle may be cut off leaving a portion of the suture exposed (Fig 5).
8. Repeat using second needle to insert the other half of the suspension suture (Fig 6-9).
9. Apply tension to the suture in place to achieve the desired lift and volume (Fig 10).
10. Trim the distal ends of the suture. It is desirable to bury the ends under adjacent, mobilized soft tissue to avoid possible erosion of the suture through the skin.
11. Repeat for each Silhouette Soft suspension suture implanted.

POST-TREATMENT RECOMMENDATIONS:

Apply cold packs immediately after procedure if required (cold packs should be wrapped to avoid direct contact with skin and insertion point). Take analgesia, e.g. paracetamol, in case of pain. Refrain from applying make-up for as long as possible (minimum 24 hours). Sleep face up, in an elevated position for 3-5 nights. Wash, shave and dry the face gently without rubbing or massaging (5 days). Avoid excessive face and neck movements (2 weeks). Avoid over-exposure to direct sunlight and do not use tanning beds (2 weeks). Avoid high impact sports e.g. running (2 weeks). Do not use saunas and hammams (3 weeks). Avoid dental surgery (3 weeks). Avoid face and neck massages and facial aesthetic treatments (4 weeks).

ADVERSE EVENTS:

Like all procedures of this type there is a possibility of adverse events, although not everybody experiences them. These adverse events include infection, minimal acute inflammatory tissue reaction, pain (which may be temporary or persistent in nature), swelling and oedema, transient haematoma or bruising and transient rippling or dimple formation. Other potential adverse events include ecchymosis, sensory/motor nerve injury, asymmetry, banding, thread migrations and palpable thread ends/knots. Material sensitivity/allergic reactions in patients following surgery may occur. The onset of any adverse event must be reported immediately. Please contact the local Sinclair representative or authorised Silhouette distributor. Alternatively send the details to quality@sinclairpharma.com

WARNINGS:

In the handling of the suspension suture and any other suture-like material, care should be taken to avoid damage from handling, if the device appears damaged prior to use, do not use. Avoid crushing or crimping damage. The superficial placement of this device may lead to all of the aforementioned side effects. Too deep placement of the suture (in the muscle) can cause pain. Do not use if the packaging is damaged or open. Although the combination use of Silhouette with other aesthetic procedures has been reported in literature, the safety of such combination treatments has not been formally established.

Single use only. Re-use of this suspension suture may result in hazardous biocontamination resulting in severe injury to the patient. Do not re-sterilize.

Discard open, unused product. Product should be disposed of in a manner that complies with the regulatory requirements of the country where it is sold.

Store between 2 °C and 27 °C in a dry place out of direct sunlight. Do not use beyond the listed expiration date on package.

If you have any complaints then please contact quality@sinclairpharma.com

9/

254mm

ARTWORK PANEL

3D Models

For more information, please call Matt Barras on
+44 01482 710380 or e-mail 3D@sonoco-trident.com

Master Ref: S/P424722

Trident Reference No: TR1893453

SKU Description: Silhouette Soft Booklet
SMS22/23/25/26/100/102 COR BKT

Brand: Sinclair

Range: Silhouette

Product: Soft

Pack Type: Booklet

Pack Size: 5x2/3x2N/A

Action: B

Date: 27/05/20

Country: Corporate

CAD Ref: VP-BKT-88.9x254mm

Manufacturing Site: Vaupell

Printer: Aloha Graphics

Substrate: Art paper

Barcode Type: N/A

Magnification: N/A

Pharmacode No/NE: N/A

Font Size (Main Body Copy): 6.00pt

Technical Et Non Printing Items

Cutter
 Guides

Colours

Process Cyan	Process Yellow	Process Magenta	Process Black
1 LTH	2 LTH	3 LTH	4 LTH

TRIDENT

Connaught House, Connaught Road,
Kingswood Business Park, Hull, HU7 3AP, England.
T: +44 (0) 1482 828100

Please note that any low resolution paper Canon colour
copies associated with this job should be referred to for
content, layout and colour separation only.

UNDER NO CIRCUMSTANCES SHOULD THIS
ARTWORK BE ALTERED WITHOUT PRIOR
PERMISSION FROM TRIDENT.

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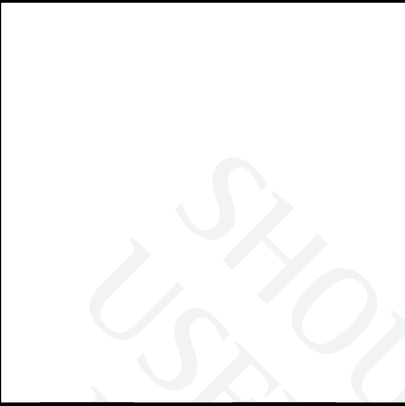
88.9mm

254mm

3D Models
For more information, please call Matt Barrass on +44 01482 710380 or e-mail 3D@sonoco-trident.com

Master Ref: SIP424722
Trident Reference No: TR1893453
SKU Description: Silhouette Soft Booklet SMS22(23/25/26/100/102 COR BKT)
Brand: Sinclair
Range: Silhouette
Product: Soft
Pack Type: Booklet
Pack Size: 5x2/3x2N/A
Action: B
Date: 27/05/20
Country: Corporate

CAD Ref: VP-BKT-88.9x254mm
Manufacturing Site: Vaupell
Printer: Aloha Graphics
Substrate: Art paper
Barcode Type: N/A
Magnification: N/A
Pharmacode No/NE: N/A

Font Size (Main Body Copy): 6.00pt

Technical & Non Printing Items
Cutter Guides

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Please note that any low resolution paper Canon colour copies associated with this job should be referred to for content, layout and colour separation only.

UNDER NO CIRCUMSTANCES SHOULD THIS ARTWORK BE ALTERED WITHOUT PRIOR PERMISSION FROM TRIDENT.

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